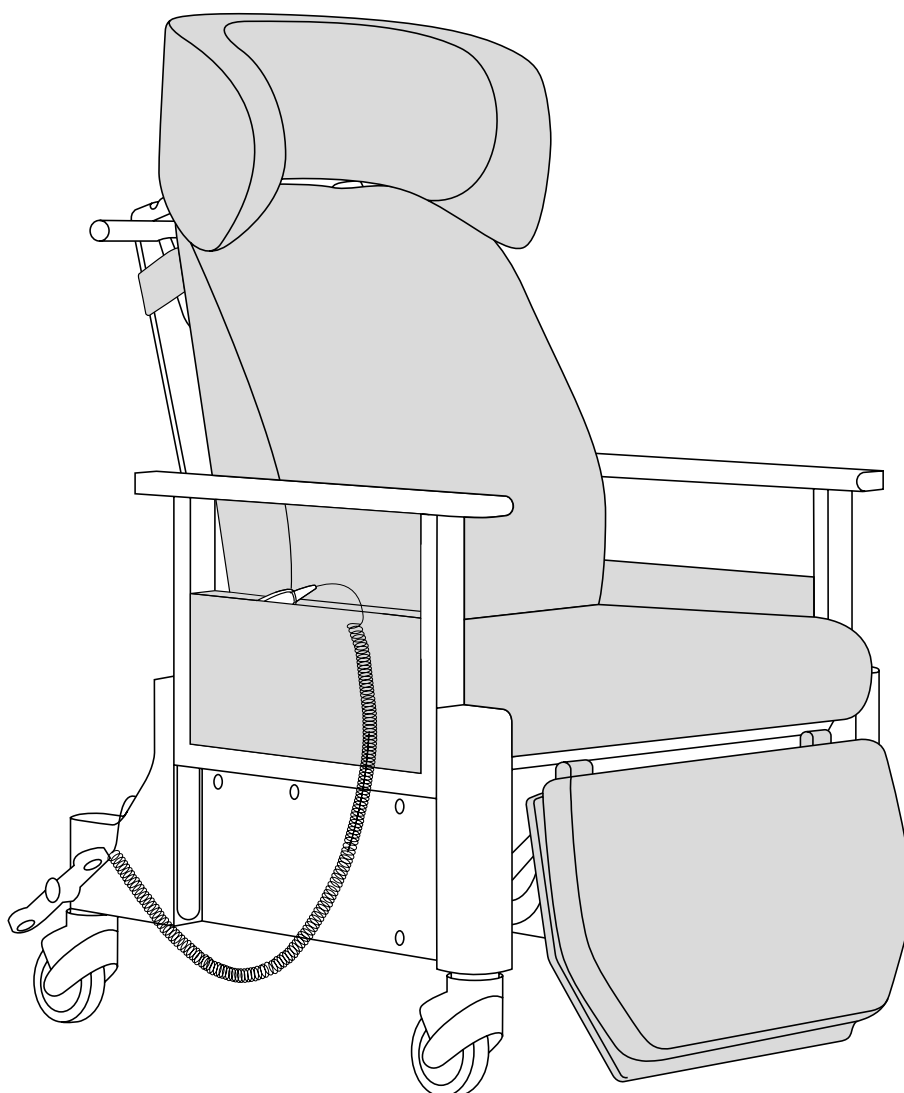
santiago

 TON

ELECTRIC RECLINER

Santiago Electric Recliner
design / René Šulc ^{CZ}

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EN Operation and Maintenance of the Santiago Electric Recliner

The electric recliner from the Santiago O2 collection is designed for professional and home-based care of people with reduced mobility. The recliner was created on the basis of findings from extensive research with seniors, people with reduced mobility and their carers. As a result, its fresh design and robust construction respond to specific use-related challenges identified by target users and ensure a high level of comfort combined with ease of operation and maintenance. The Santiago collection has been tested with the same stress and mechanical tests applied to all TON products. We are therefore confident that it will serve you well beyond its five-year warranty – especially if you follow the guidance outlined in this operation and maintenance manual.

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EN Before first use

The recliner is intended to be used on a smooth and flat surface. If you need to lift the recliner, do not lift by its armrests, rear handrail or upholstery – only lift the recliner by the bottom of the side panels. Do not keep the recliner in close proximity to a heat source or in direct sunlight, as this can cause warping of the wood or discolouration.

After you unpack the recliner, charge the battery with the power cable stowed behind the flip-down panel in the rear of the chair. The first charge should take at least 8 hours. The recliner should be placed within easy reach of a power source – it is not advisable to use an extension cord.

The recliner control unit is permanently attached by a cable running from the flip-down panel located in the rear of the chair, and does not need to be disconnected. It is good to store the control unit inside the designated tab in the side pocket. The side pocket can be removed for cleaning as needed.

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! After charging, please wind the power cable onto the power cable organiser and secure the plug. This will prevent the cable from being damaged during reclining or chair transfer.

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EN Reclining

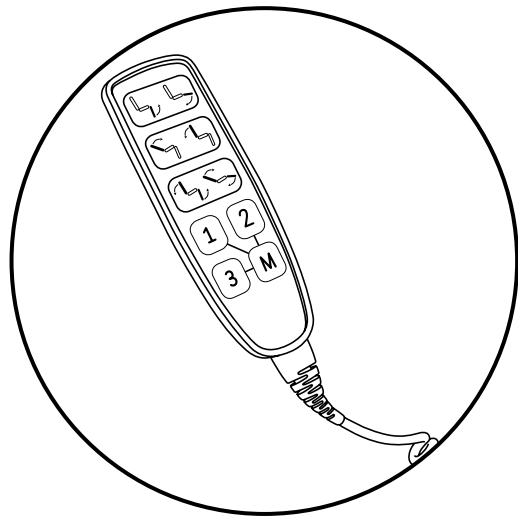
The recliner control unit allows you to recline the chair into its maximum reclined position and to return to the resting position. The backrest and the footrest can be controlled independently. The control unit also enables you to store three pre-set positions. To store a pre-set position, first use the control unit to move the chair into the desired position and then press the M key and simultaneously the desired pre-set number (1, 2 or 3). The chair will automatically move into any of the three pre-set positions after you press the relevant position number.

Please take care of the safety of small children and animals when reclining the chair.

If the chair is reclined on a daily basis, it is best to recharge it overnight.

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! Please observe the recommended load limit noted on the control box located at the bottom of the reclining mechanism. If the load limit is exceeded, there is a risk of overheating and damage to the reclining system. Unless indicated otherwise, load can be exerted on the reclining system for a maximum of 10% of total use time: in other words, 2 minutes of active use (reclining back and forth) followed by 18 minutes of rest. If this limit is not observed, the recliner will wear quickly.

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EN Battery

The recliner uses a waterproof lithium-ion battery with a built-in safety charge system. Before first use, charge the battery for a minimum of 8 hours. The recliner can be connected to a power source for several days without any risk of overheating. Do not use the recliner in an area where the battery may be exposed to flammable anaesthetics in combination with air, or to oxygen or nitrogen oxide (laughing gas). Excessive heat or close proximity to a heat source can also affect the battery: do not expose the battery to high-intensity UV lamps, use it near an open flame, or use or store it at temperatures higher than 50 °C. If you want to clean the battery, do not use aggressive chemicals, thinners, acids or alkaline fluids, do not use a high-pressure cleaner, and do not submerge it in water. Plastic components may be damaged in contact with mineral oils. Please recycle an expired battery at a designated recycling facility. The battery warranty is two years.

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EN Movement

Move the recliner by holding only the rear handrail, not the headrest. When moving a seated person, make sure that the footrest is raised high enough to avoid dragging the person's feet on the floor. Also ensure that the control unit and/or the control cable are safely stowed to avoid possible damage to the control mechanism.

The headrest can be adjusted mechanically, both in terms of height and angle.

The armrests have two positions: top (resting) position and low position, which enables easy transfer of a person to and from the chair. To lower the armrest, first raise it lightly and move it to the side, then lower it. Reverse movement will bring the armrest back to the top position.

Manoeuvring of the chair is controlled with the help of plastic levers located above the rear wheels. Position 1 serves as a brake; position 2 locks the wheels in a straight line, making transport easier over long distances; position 3 allows the wheels to swivel for ease of manoeuvrability.

The removable tray table slides onto the inside of the armrests and is secured in place with plastic clips. After use, hook the tray table on the rear handrail and secure it with the belt and the magnet located at the bottom of the rear of the backrest.

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EN Upholstery

The seat and the footrest contain PUR foam. The backrest and the headrest contain injected foam that has been moulded to ergonomic precision. The recliner can be upholstered in a wide range of fabrics, many of which are easy to maintain and clean. For fabric maintenance follow the manufacturer's guidelines.

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EN Additional information

The recliner mechanism and drive unit do not require any special maintenance. If dirt should become lodged between the wooden side panels and the seat or the backrest, it can be removed with the help of a vacuum cleaner with a narrow attachment.

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! The control unit, the bottom of the rear of the backrest, and the footrest upholstery all contain magnetic parts.

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